



Remedy Specialized DriGo-HP

Antibacterial Moisture Wicking Sheet


Product Information:

- For use in skinfolds and other skin-to-skin contact areas to aid in moisture management, odor control, and friction-reduction.
- Can be used along with Anti-Fungal Treatments.
- Effective for up to 5 days.




How to Use:

1. Wash skin gently and pat dry, do not rub.
2. Cut the appropriate size of fabric with clean scissors.
3. Place a single layer of fabric in the skinfold, placing one edge of the fabric into the base of the fold. Avoid wrinkles and keep the fabric flat, leaving at least 2 inches exposing outside the skin fold for moisture wicking.

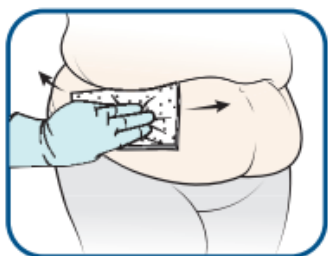

CLEAN & DRY SKINFOLD

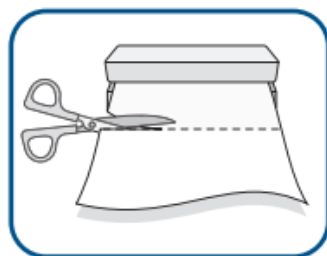

CUT SHEET TO SIZE



LIFT & INSERT SHEET



MOISTURE DRAINS

Removal:

- Dri-Go can be left in place for up to 5 days, depending on soiling, odor, amount of moisture and condition of the skin.
- To remove, gently separate the skinfold and lift away the fabric.
- Single use only, discard each cut piece after use.